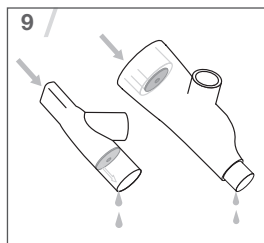
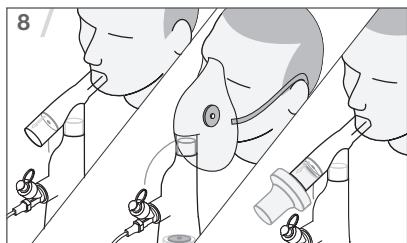
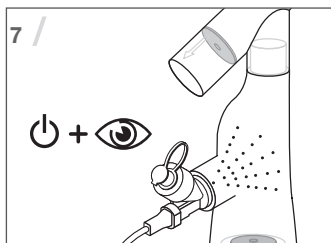
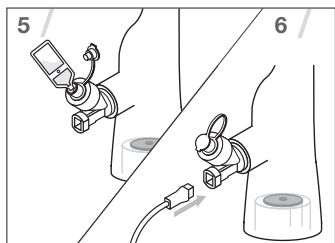
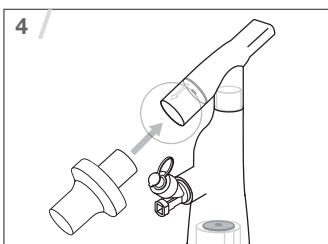
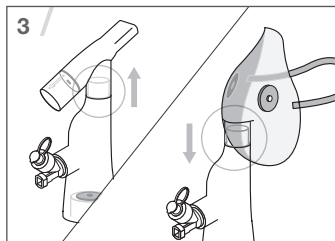
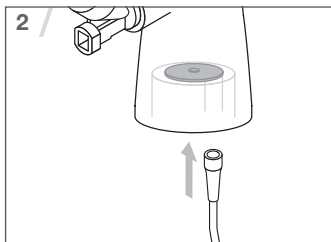
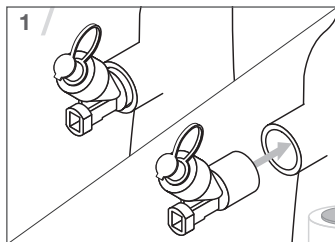
Aerogen®

Aerogen® Ultra

Instruction Manual

The Aerogen® Ultra is an accessory specific to the Aerogen Solo nebuliser. It facilitates intermittent and continuous nebulisation, with optional supply of supplemental oxygen to paediatric and adult patients via mouthpiece. The device can alternatively be used with a face mask, as supplied.

It is a single patient use device which is qualified for 20 intermittent use treatments (at a rate of four 3 ml doses per day over 5 days) or 3 hours of continuous use.

For continuous use, the Aerogen Ultra can be used in conjunction with the Aerogen Solo Continuous Nebulization Tube Set. See Aerogen Solo System Instruction Manual.

Optimal aerosol delivery is achieved with valved mouthpiece or valved aerosol face mask with low/no oxygen flow.

The Aerogen Ultra is for use in spontaneous and conscious breathing patients only.

The Aerogen Ultra, Mouthpiece, Aerogen Solo nebuliser, oxygen tubing and I-Guard™ Aerosol Mask are non-sterile.

The filter is not supplied by Aerogen. The image of the filter has been included for demonstration purposes. It is the responsibility of the clinician to determine if a filter is required and the type of filter selected for use (Viral/Bacterial) in conjunction with the Aerogen Ultra. When selecting a filter for attachment to the Aerogen Ultra mouthpiece, Aerogen recommend using a filter with a minimum efficiency rating of 99.9% (Bacterial) or 99.8% (Viral).

If using the Aerogen Ultra in conjunction with a filter, refer to the filter manufacturer's Instructions for Use for information including filter disposal.

For disposal of the Aerogen Ultra, Mouthpiece, Aerogen Solo nebuliser and oxygen tubing, refer to hospital or institutional protocol, for disposal of the I-Guard™ Aerosol Mask refer to manufacturer's instructions.

The following devices are governed under CE0050	The following devices are not governed under CE0050
<ul style="list-style-type: none"> • The Aerogen Ultra & oxygen tubing • The Aerogen Ultra Mouthpiece • The Aerogen Solo Nebuliser 	<ul style="list-style-type: none"> • The I-Guard™ Aerosol Mask • The Filter

Instructions for Use - Refer to Figures 1 - 9

Inspect for device integrity and correct valve placement prior to use.

1. Insert the Aerogen Solo nebuliser firmly into Aerogen Ultra in orientation shown.
2. If supplemental oxygen is required, firmly attach oxygen tubing to Aerogen Ultra.
Note: Oxygen flow rate should be set between 1-6 LPM.
3. If a face mask is required, remove mouthpiece where applicable and attach face mask to Aerogen Ultra.
Note: When using an open face mask, a minimum oxygen flow of 1 LPM is required.
4. If use of a filter is required, the Aerogen Ultra Mouthpiece has a 22mm(F) ISO 5356-1 connection port to facilitate the attachment of a ISO 5356-1 compliant filter port.
5. Add prescribed medication to the Aerogen Solo nebuliser as instructed in Aerogen Solo System Instruction Manual.
6. Connect cable to Aerogen Solo.
Note: Refer to Aerogen Solo System Instruction Manual for intermittent and continuous nebulisation set-up and operation instructions.
7. Power on and observe Aerogen Ultra to confirm aerosol is visible.
8. Introduce the Aerogen Ultra to patient and observe aerosol flow to ensure correct operation.
9. Remove excess rainout from the Aerogen Ultra periodically (hourly with continuous nebulisation). To ensure optimum performance of the Aerogen Ultra, remove any residue by rinsing through with sterile water, shake off excess and allow to air dry.

Warnings

Performance

Do not use with a closed face mask.

When using with an open face mask, always use supplemental oxygen flow of 1-6 LPM.

Performance of Aerogen Ultra may vary depending upon the type of drug and Aerogen Ultra configuration used.

Do not exceed recommended oxygen flow for system.

Visually check the Aerogen Ultra post-rinsing to ensure that valves have not become dislodged.

When connecting a 22mm(M) breathing system filter to the Aerogen Ultra Mouthpiece, ensure that the filter does not occlude the exhalation valve of the mouthpiece.

Change the filter as per manufacturer instruction or more frequently if it becomes obstructed.

The Aerogen Ultra, when used in combination with the Mouthpiece, Aerogen Solo nebuliser and a filter (with a recommended minimum efficiency rating of 99.9% (Bacterial) or 99.8% (Viral)) has the ability to reduce but not eliminate the risk of transmission or acquisition of an infectious agent by healthcare workers or others.

General

To be used by trained medical personnel only.

Use only with recommended components.

Ensure tubing is safely orientated to prevent strangulation hazard.

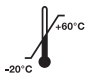
Ensure oxygen connection port or tubing is not occluded.

Do not use the Aerogen Ultra without a mouthpiece or face mask.

Do not autoclave any component of the kit.

Do not cover Aerogen Ultra valves during use.

Symbols Glossary

Symbol	Title / Description (if applicable)
	The CE mark indicates conformity to the Medical Device Directive 93/42/ EEC as amended by 2007/47/EC. The 0050 number under the CE mark denotes the Notified Body identification number (0050 -NSAI).
	This product is supplied non-sterile.
	Consult instructions for use: Indicates the need for the user to consult the instructions for use.
	Transient Temperature Limit: -20 °C to +60°C. Indicates the temperature limits to which the medical device can be safely exposed.

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© 2023 Aerogen Ltd.
Aerogen Ltd.