

## Disposable EEG Sensor Adult for measurement of the bispectral index

MedkerTech<sup>®</sup>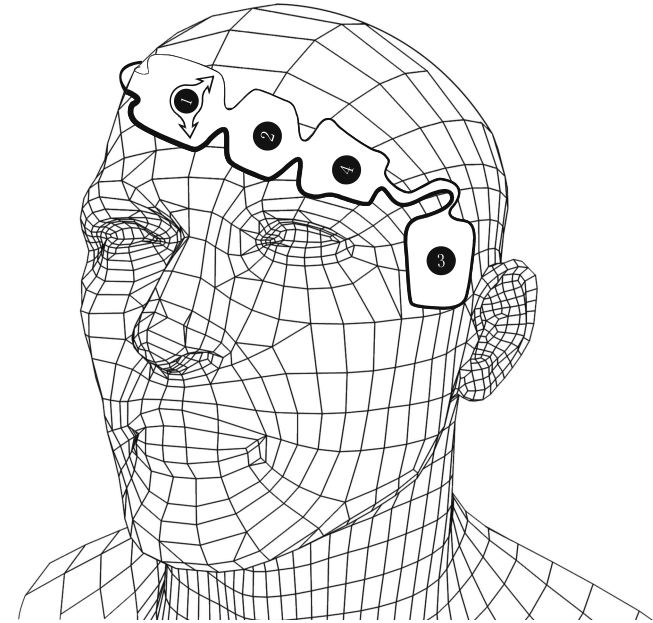

EN	Instructions for Use	2
ES	Instrucciones de uso	3
IT	Istruzioni per l'uso	4
NL	Gebruiksaanwijzing	5
CS	Návod k použití	6
DA	Brugsanvisning	7
SV	Navodila za uporabo	8
HU	Használati útmutató	9
RO	Instructiuni de folosire	10
HR	Upute za korištenje	11
SR	مادخت سالا تامي لعت	12
RU	Инструкции по использованию	13
EL	Οδηγίες χρήσης	14
KO	사용 지침	15
JP	使用説明書	16
TR	Kullanım için talimatlar	17
SL	Instrukcie na používanie	18
NO	Instruksjoner for bruk	19
FI	Käyttöohjeet	20
PL	Instrukcja użycia	21
PT	Instruções de uso	22
DE	Gebrauchsanweisung	23
FR	Instructions d'utilisation	24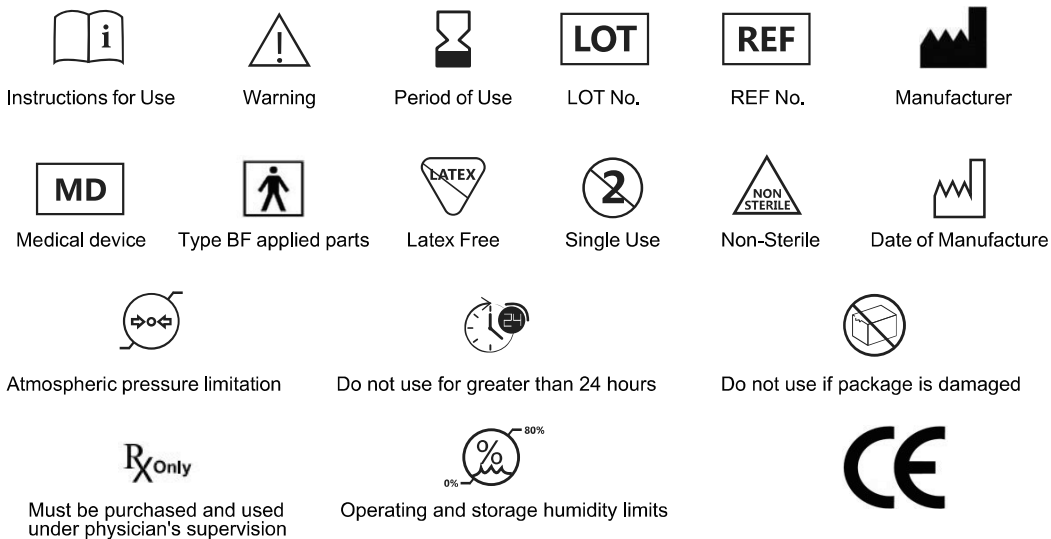

Manufacturer: Medker Medical Electronic Tech (Shenzhen) Co., LTD  
 Production address: Rm 501, Building A, Yeming Mould Industrial Park, No. 19 Baoshan Road, Tianliao Community, Yutang Street, Guangming District 518132, Shenzhen  
 Contact information: 0086-755-23406503  
 E-mail: info@medker.cn  
 Website: www.medker.cn

EU Representative: Sungo Europe B.V.  
 Address: Olympisch Stadion 24, 1076DE Amsterdam, Netherlands  
 Contact: +31(0)202111106  
 E-mail: ec.rep@sungogroup.com

**EN:**

1. Intended Use:

The disposable EEG Sensor is used together with the Anesthesia Depth Monitoring for collecting and transmitting EEG biological signals to monitor the anesthesia depth of patients.

2. Warning:

To reduce the hazard of burns during use of brain - stimulating devices ( e.g., transcranial electrical motor evoked potential), place stimulating electrodes as far as possible from the disposable bispectral index sensor and place sensor according to instructions.


3. Cautions:

Patient position may increase the risk of skin irritation on the forehead, with patients in prone position, consider minimizing pressure on the sensor.


4. Note:

Do not use if sensor is dry, to avoid dry out, do not open pack until ready for use. Due to intimate skin contact, reuse may pose risk of infection. If skin rash or other unusual symptoms develop, stop use and remove. Limited to short - term use (maximum 24hours). Do not cut sensor, as this will result in improper operation. Upon removal, slight redness of skin may occur and typically resolves within a short period of time.

5. Using Instructions :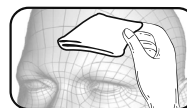

1. Wipe the skin with wet gauze by alcohol and dry

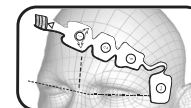2. Place the sensor diagonally on the forehead as shown


2.1 place ① position 5cm directly above the bridge of nose in the center of the forehead


2.2. Place ④ directly above the eyebrows


2.3. Place ③ in the temple position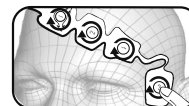

3. Squeeze sensor edges to ensure adhesion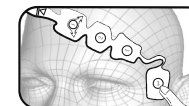

4. Press ①②③④ sequence for 5 seconds to ensure adhesion

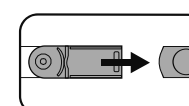5. Insert the sensor plug into the interface cable

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